

A woman with long blonde hair, wearing a white strapless dress, stands in a lush, green forest. She is adorned with a garland of green leaves and small purple flowers around her waist. The scene is filled with soft, glowing light and numerous small, white, star-like sparkles scattered throughout the air. Several purple butterflies are seen fluttering around her. The background is a dense, out-of-focus forest with tall trees and vibrant green foliage.

Bring out  
your inner  
beauty

  
TEOXANE

THE EXCELLENCE OF SWISS SCIENCE

---

## Everybody is speaking about hyaluronic acid! But how exactly does it work?

---

Hyaluronic acid (HA), which is naturally present in our body, performs a fundamental role in keeping the skin in good health. This essential element of the dermis retains water like a sponge so as to maintain its hydration.

## SKIN UNDERGOES CHANGES WITH THE PASSAGE OF TIME

The main reasons for the ageing of the skin include excess of oxidants (free radicals)\* and progressive loss of intrinsic HA.



With age, we can lose up to 2/3 of our hyaluronic acid pool.<sup>1</sup> At 35 years of age the skin's thickness has decreased on average by about 22%.<sup>2</sup>

As it gets thinner, the skin becomes more fragile, dehydrates and its elasticity diminishes.

*Just like an apple that dries, skin that is deprived of its sources of hydration will lose its shine with the passage of time.*

\*An unhealthy lifestyle associated with other factors, such as an unbalanced diet and cigarette smoking, affects the skin's appearance and quality.

---

## It's not inevitable! Although our skin undergoes changes we can take action to stimulate its beauty!

---

## AN ALTERNATIVE FOR PREVENTION

The hyaluronic acid injections are designed to prevent premature ageing of our skin and to boost the radiance of our face.<sup>3</sup>

## A COMPENSATION FOR THE SKIN

The hyaluronic acid injections compensate the natural loss of this substance and stimulate the auto-production of collagen<sup>4</sup> to help:



**The efficacy of injectable hyaluronic acid enables us to push ever forward into the future the need for anti-ageing surgery.<sup>5</sup>**

*Approximately 80% of facial skin ageing is attributed to UV-exposure.<sup>6</sup>*

# Redensity [I]

## BEAUTY BOOSTER

### Holding back the appearance of the first wrinkles

Your skin is starting to change? Would you like to keep it replenished, hydrated and beautiful for longer? Protect and stimulate it with Redensity [I].<sup>7</sup>

### A UNIQUE AND PATENTED FORMULA

A high concentration of natural and free hyaluronic acid + a combination of anti-oxidants, amino acids, minerals and vitamin, all of which are naturally present in our skin.

#### HYDRATION

HA

ARGININE

#### ANTIOXIDANT PROTECTION

HA

GLUTATHIONE, N-ACETYL-L-CYSTEIN,  
ALPHA-LIPOIC ACID



#### TISSUE RESTRUCTURATION AND CELL REGENERATION

HA

LEUCINE, ISOLEUCINE, VALINE, GLYCINE, LYSINE,  
THREONINE, PROLINE, VITAMIN B6, ZINC, COPPER

Redensity [I] is a non cross-linked HA gel, fluid, which allows a smooth application by micro-injections at the chosen areas.

TEOSYAL Redensity® [I] is a product of the TEOSYAL® PureSense range (medical devices, class III, CE mark) for medical professional use only.

### Smoothing out a mature skin

Is your skin more creased and thinner than before? Would you like to extend and perfect the filling effect? Hydrate and redensify your skin with Redensity [I].

### WHEN & WHERE SHOULD IT BE USED?<sup>8</sup>

**For your personal protocol, ask your practitioner who will recommend the most optimal treatment for your needs.**

- Regularly as core treatment to prevent cutaneous ageing
- In alternation with or as a complement to other treatments
- On specific chosen occasions to rehydrate and give a new glow to your skin (for example, before/after prolonged sun exposure...)

**For the face, neck and neckline.**



**For an optimal result, it is recommended to follow the complete protocol to extend the effects of the treatment and obtain long-lasting results:**

- for an optimum treatment: 3 sessions at 3-week intervals for immediate and cumulative results
- for a maintenance treatment of the results, as soon as the need is felt: 1 session usually 3 times a year

# Redensity [I]

## BEAUTY BOOSTER

Skin quality improvement\*...

...approved by patients\*

IMPROVEMENT:

SKIN GLOW  
**+60%**

ELASTICITY  
**+56%**

TEXTURE  
**+59%**

HYDRATION  
**+61%**

# 91.7%

OF PATIENTS WERE SATISFIED  
WITH THEIR TREATMENT

**91.7%**  
ARE READY  
TO REPEAT  
THE TREATMENT

**90.3%**  
WOULD  
RECOMMEND  
THE TREATMENT

«I instantly recovered a healthy glow,  
everyone told me so.»  
*Olga, 53*

«The result is natural and this is  
what I was looking for.»  
*Lisa, 47*

«My impression is that  
my face is firmer,  
my skin replenished...  
I feel better about myself.»  
*Anna, 35*

«Redensity [I] is like a super-cream  
that plumps the skin from the inside.  
The effect is immediate and long-lasting.»  
*Peggy, 42*

\* Study report Teosyal® PureSense Redensity [I]. Dr Knoll, Dr Vivier, Dr Carre, Dr Micheels. for the face indication, 9 weeks after the first injection session. The protocol consisted of 3 tolerance : the main reported side-effects were transient rashes for the face and transient

May 2013. Results from the doctor's evaluation of the global aesthetic improvement injection sessions at 3-week intervals. This study showed a very good treatment rashes, pain, ecchymosis and itching for the neck and neckline injections.





TEOSYAL PureSense  
**Redensity [I]**  
BRING OUT YOUR INNER BEAUTY

### WHAT HAPPENS DURING THE SESSION?

- A session consists of intra or hypodermic micro-injections given with a fine needle of very small quantities of Redensity [I] along the lines of wrinkles and evenly spread over the surface of the whole face. In this way the Redensity [I]'s HA and nutrients arrive at the heart of the skin.
- The injection may be «manual», with syringes and needles, but also «electronically assisted» for more comfort (e.g. Teosyal® Pen).
- A session lasts about 20 minutes.

### WILL I BE ABLE TO START WORKING AGAIN IMMEDIATELY AFTER A SESSION?

A very slight lack of comfort and a few small spots may appear at the injection points\*. These inconveniences generally disappear quickly after injection.



\* There are potential side effects related to the implantation of this device, occurring erythema, redness, papule. The patient should inform the practitioner of any secondary

TEOSYAL®PEN is a motorized and cordless device for hyaluronic acid injections in order to is a regulated health product which bear, under this regulation, the CE mark (CE0434). For

### ARE THE RESULTS IMMEDIATELY VISIBLE?

The results are seen immediately at the first session. The skin has a better colour, is less creased, better hydrated and becomes more supple. Gradually, from one session to the next, the complexion looks increasingly radiant and the skin becomes firmer.<sup>7</sup>

### DOES IT REQUIRE PAIN RELIEF?

The micro-injections are shallow and performed with very fine needles. For better comfort throughout the injection session Redensity [I] contains lidocaine which is a local anaesthetic widely used in the world. For enhanced comfort it is possible to apply an anaesthetizing cream before treatment.

### HOW LONG DOES THE EFFECT LAST?

Hyaluronic acid is a resorbable molecule, its effects are temporary and depend on several factors: skin type, age, treated area, injection technique...

For an optimum result, an initial protocol of 3 sessions and a maintenance session 2-3 times a year are recommended.

immediately following injection or after a delay. These include, among others : oedema, effect other than those described above or any effect persisting beyond one week.

correct wrinkles and depressions - Manufactured by Juvaplus SA. This medical device (class IIa), more information, please consult your doctor and TEOSYAL®PEN's instructions for use. »



Located in Geneva, Switzerland, TEOXANE Laboratories are specialized in the design and manufacture of hyaluronic acid-based products for aesthetic medicine.

# TEOSYAL PureSense Redensity [I] BEAUTY BOOSTER

Teosyal® PureSense Redensity [I] is indicated for the prevention of wrinkles and rehydration of the face, neck and neckline by injection by a health professional. Its main components are hyaluronic acid (15mg/ml) and lidocaine hydrochloride (3mg/g). For more information, consult your doctor. This is a regulated health product (class III medical device, CE 0086).

#### References :

1. Labat-Robert J. et al. Aging of the extracellular matrix. *Médecine & Longévité*. 2012 March; 4(1): 3-32.
2. Farage MA, Miller KW, Elsner P, Maibach HI. Characteristics of the Aging Skin. *Adv Wound Care (New Rochelle)*. 2013 Feb;2(1):5-10.
3. Baspeyras M, et al. Clinical and biometrological efficacy of a hyaluronic acid-based mesotherapy product: a randomised controlled study. *Arch Dermatol Rés.* 2013 Oct;305(8):673-82.
4. Anderégg U, et al. More than just a filler - the role of hyaluronan for skin homeostasis. *Exp Dermatol*. 2014 May;23(5):295-303.
5. Data from SNDV (Syndicat National des Dermatologues-Vénérologues)
6. Papakonstantinou E, et al. Hyaluronic acid: A key molecule in skin aging. *Dermatoendocrinol*. 2012 Jul 1;4(3):253-8.
7. Data from AFME (Association Française de Médecine morpho-Esthétique et anti-âge)
8. Teosyal® Puresense Redensity [I] instructions for use.

500310/00, October 2015



TEOXANE

THE EXCELLENCE OF SWISS SCIENCE

TEOXANE SA - Les Charmilles  
Rue de Lyon, 105 - CH 1203 Geneva  
www.teoxane.com



Visit our website:  
[www.lookyounger.com.au](http://www.lookyounger.com.au)  
Email [sales@lookyounger.com.au](mailto:sales@lookyounger.com.au)  
or call AUS 1800 633 830  
NZ 0800 445 83